

# I N S T R U C T I O N S



## Extra Wide Tall-Ette® Toilet Seat with Steel Frame

72588-2000

1. Insert the two safety bars into the four holes in the top of the plastic seat.
2. Push the four rubber feet onto the ends of the safety bars.


**Note:** Dipping the rubber feet in soapy water will make them slide on easier.

### To install on toilet:

1. Loosen nuts on the screws that hold the toilet seat and lid to the toilet.
2. Lift the toilet seat to a raised position and pull up on it so there is a small gap between the seat and the toilet.
3. Slide the Lok-in-EI® Bracket tongue under the seat bracket so that the middle section fits between the toilet seat screws and the two narrow sections fit outside the screws.
4. Tighten the nuts on the screws to secure the seat and bracket. The Tall-Ette® should be in contact with the bowl. The bottom flange fits inside the rim of the toilet bowl.

### Caution:

- Do not use this model without the Lok-In-EI® Bracket.
- Do not remove the Lok-In-EI® Bracket.
- Do not use the Tall-Ette® Elevated Toilet Seat if it does not fit into the toilet bowl rim.

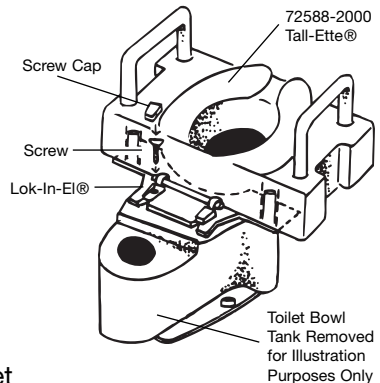

## Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)